DIRECTIONS FOR USE

see www.insafedental.eu/#how-to-use
or scan QR Code for demo video

YOUNG®
INNOVATIONS

Index

EN	Instructions for use	3
DA	Brugsanvisning	13
DE	Gebrauchsanweisung	23
ES	Instrucciones de uso	33
FI	Käyttöohjeet	43
FR	Notice D'utilisation	53
IT	Modalità D'impiego	63
NL	Gebruiksaanwijzing	73
NO	Bruksanvisning	83
PL	Instrukcja Użytkowania	93
SV	Bruksanvisning	103

Instructions for use

Sharps Container Fitting


[1a]

Remove protective covers from adhesive pads.

[1b]

Position base on a smooth clean worktop and push down firmly to secure.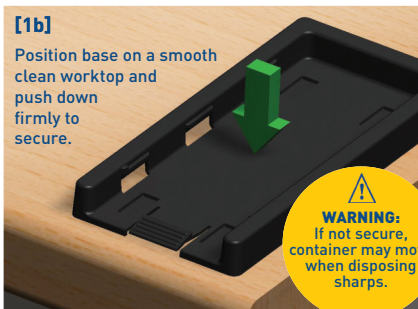


WARNING:

If not secure, container may move when disposing sharps.

[1c]

Slide sharps container into base until it clicks into place.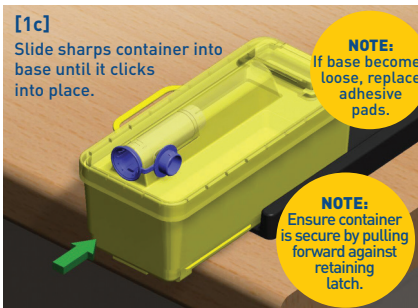

NOTE:

If base becomes loose, replace adhesive pads.

NOTE:


Ensure container is secure by pulling forward against retaining latch.

2

Needle Loading

[2a]


Align hole on needle adaptor with slot on body and push fully home.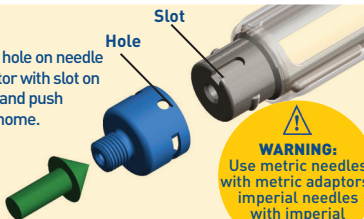

WARNING:
Use metric needles with metric adaptors/
imperial needles with imperial adaptors.

[2b]

Remove rear needle cover and screw needle fully on.

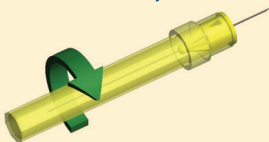NOTE:

Compatible with needles of 38 mm length or less.

[2c]

Turn sleeve collar fully directional such that the arrows are misaligned.

[2d]

Slide sleeve forward to fully extended position.

WARNING:
Do not re-use adaptor! Risks include cross-infection and alteration/degradation of device material and function.

[2e]

Turn sleeve collar fully such that arrows are aligned to lock.

WARNING:


Ensure that the sleeve lock collar is fully activated. If it is not then accidental needle stick injury may occur.

Cartridge Loading

[3a]

Pull plunger back against spring and hold.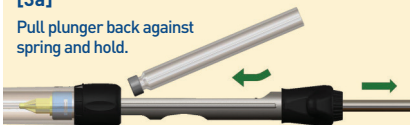

[3b] Slide cartridge into syringe body opening and release plunger to secure.

NOTE: Use only with 1.8 mL or 2.2 mL dental cartridges dimensionally compatible with ISO 9997-compliant syringes. Compatibility has been verified with standard cartridges consistent with ISO 11499 dimensions.

Syringe Use

- > Grip sleeve lock collar and turn until arrows are misaligned **[2c]**.
- > Slide sleeve fully back.
- > Turn lock collar until arrows are aligned **[2e]**.
- > Give injection as normal.
- > Grip sleeve lock collar and turn until arrows are misaligned **[2c]**.
- > Slide sleeve fully forward **[2d]**.
- > Turn lock collar until arrows are aligned **[2e]**.
- > Repeat above as required for further injections.
- > Load new cartridge as shown in section 3.

5 Changing Cartridge

- > With sleeve in forward locked position hold plunger stem **[3a]**, pull back against spring and hold.
- > Grip cartridge with index finger and thumb through upper and lower syringe body apertures.
- > Push cartridge upward and back to remove.
- > Load new cartridge as shown in section 3.

6 Needle Removal

[6a] With sleeve locked in the forward position, enter the syringe into the sharps container lid aperture.

WARNING:
In order to avoid trapping fingers, please ensure that they do not protrude beyond the 'T' handle.


[6b] Holding syringe 'T' handle, and exerting slight forward pressure, turn the syringe anti-clockwise a small amount to unlock, and push syringe forward into container.

WARNING:

The syringe must be entered fully into the sharps container to engage the 'lock'. Failure to achieve this may cause accidental needle stick injury.

[6c] Push syringe fully forward - syringe is now 'locked' into container.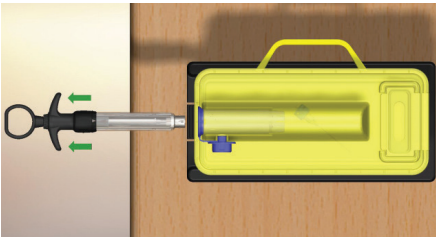

[6d] With index and second fingers under 'T' handle, pull syringe sharply backwards to remove the needle.

Sharps Container Removal

- When the container has been filled to the maximum indicator line disposal is required.
- Close the forward and rear container apertures and push fully home to lock.
- Press down on the lower latch feature on the base and pull the container forward and away from the base.
- Holding the container by the attached handle remove it to a safe area for disposal collection. For more information on safe disposal, see warning label on inSafe Sharps container.
- Load an empty container onto the base as shown in section 1.

8

Cleaning and Sterilisation

The syringe is supplied non-sterile and should be autoclaved before first use.

Preparation

- > Manually remove all visible soil, such as saliva, from the device with a disposable cloth or paper wipe.
- > Unscrew locking nut to detach plunger from body housing.

Cleaning – Manual

- > Manually clean at 36 °C using a solution of Ruhof Endozyme AW Plus detergent in a ratio of 4mls: 1 liter of water (or a pH neutral enzymatic detergent prepared per manufacturer instructions).
- > Use a soft bristle brush to remove soil.
- > Rinse thoroughly with water.

Cleaning – Automated

- > Cold pre-wash: 2 minutes.
- > Wash stage: 2 minutes using Serchem Medical Phoenix neutral detergent (or pH balanced enzymatic detergent for use in automated washer or ultrasonic machine according to manufacturer instructions) or Serchem pH Plus Detergent (or alkaline detergent for use in automated washer or ultrasonic machine according to manufacturer instructions).
- > Rinse stage: 2 minutes.
- > Disinfection stage: 80 °C for minimum 10 minutes/AO Value 600.
- > Drying: 90°C.

Sterilisation

- > Following cleaning, as above, with the syringe sleeve locked in the forward position, place both syringe parts in a suitable autoclave pouch.
- > Place in an autoclave and sterilise using a standard steam autoclave cycle to 134°C for a minimum of 3.3 minutes.
- > To dry, use the drying cycle of the steam autoclave. Set cycle for 30 minutes. Do not exceed 137°C.
- > After removing device from autoclave, allow it to return to room temperature before use.
- > Do not let device remain in pouch under wet/damp conditions.

Reprocessing Limitations:

Syringes have been tested to withstand 100 reprocessing cycles per ISO 9997 without deterioration in performance or signs of corrosion. End of life is determined by wear and damage in use as well as reprocessing cycles. Inspect for defects before use.

PLEASE NOTE:

Do not remove and load a second needle onto the same syringe during a patient procedure. If a new needle is required then use a clean sterilised syringe.

The instructions and cleaning products provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result.

The needle adaptor is single use only. Do not clean. Sterilization is not required under normal use. If local regulations mandate sterilization prior to use, follow the validated steam cycle below. After use, immediately dispose of the adaptor. Re-use is not permitted.

Sterilisation

- > Place adaptor in a suitable autoclave pouch.
- > Place in an autoclave and sterilise using a standard steam autoclave cycle to 134 °C for a minimum of 3.3 minutes.
- > To dry, use the drying cycle of the steam autoclave. Set cycle for 30 minutes. Do not exceed 137 °C.
- > After removing device from autoclave, allow it to return to room temperature before use.
- > Do not let device remain in pouch under wet/damp conditions.

9

Changing Outer Sleeve

[9a]

Unlock sleeve and move to position shown.

[9b]

Insert suitable tool into hole on collar, push down and separate collar from sleeve.

[9c]

Remove small locking piece and slide sleeve off syringe body.

[9d]

Replace sleeve with new part, insert locking piece, push tab inward and slide collar into place.

[9e]

Check operation of lock collar and sliding action.

10 Syringe Types

Standard Syringe

- > Syringe option with 'T' plunger handle and plain plunger end is non-aspirating.

Aspirating Syringe Cartridge Seal Type

With the syringe needle and cartridge loaded as above push lightly on the plunger to engage the plunger end into the cartridge bore.

- > Prior to giving the injection expel a small amount of anaesthetic.
- > Enter the needle into the tissue of the desired operation site.
- > When the required needle penetration depth is reached draw back the syringe plunger slightly (2mm) and check for signs of blood in the forward (needle distal end) area of the cartridge.
- > If blood is evident withdraw the needle and re-enter the tissue at a different site. Repeat the above until no blood is seen.
- > At that point administer the amount of anaesthetic required.

Self Aspirating Cartridge Type

Self Aspirating (T Type)

Self Aspirating (Thumb Ring)

For use with self-aspirating cartridges

Harpoon Type

Harpoon Aspirating (Thumb Ring)

IMPORTANT NOTICE - PLEASE READ

InSafe local anaesthetic Syringe System is intended for use solely by appropriately qualified or trained dentistry professionals and staff. The inSafe Syringe System is intended for use in administering local anesthetics to specific sites during dental procedures and for safely removing contaminated needles without direct contact.

To avoid physical harm to patients, or damage to equipment and/or malfunction, use only adapter and other syringe components made by Young Innovations for the InSafe Syringe System, and always follow the manufacturer's instructions and recommendations. Use only with 1.8 mL/ 2.2 mL dental cartridges in compliance with ISO 11499 and needles shorter than 38 mm. Failure to do so will be entirely at the user's own risk and Young Innovations cannot be responsible for any loss or damage arising in any way as a result of a failure to follow such instructions and recommendations.

Any express or implied warranty is excluded to the extent permitted at law.

Report any serious injury in relation to this device to the manufacturer and the competent authority of the Member State in which the incident occurred.

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Young Microbrush Ireland Ltd
MDSS GmbH
MDSS CH GmbH
Astek Innovations Ltd.